



**SitnStand**  
Portable Smart Rising Seats



Portable lifting  
cushion  
**Réf : 825 100**



Wheelchair  
lifting cushion  
**Réf : 825 102**

## Improve your quality of life

The SitnStand lifting cushion allows to stand up and sit down again safely and independently.

This equipment is also a valuable aid for carers and caregivers in preventing musculoskeletal disorders.

Very easy to use, thanks to a controller and a small battery-powered compressor that inflates the seat, with adjustable height as required.

This cushion is available in two versions:

- Suitable for any style of chair
- for wheelchairs

### Technical characteristics :

- Nominal operation time (inflate/deflate) : ~ 20 sec
- Control mechanism : push button ↑ and ↓
- Air capacity : ~ 1.7L/sec
- Inflation levels : 3 for the wheelchair version and 4 for the seat version.
- Battery type : Lithium Ion 14,4V - 2,6 Ah
- Battery life : 300 cycles
- Inflation/deflation cycles between charges : ~ 50 cycles



## DIMENSIONS AND WEIGHT :

- Seat footprint deflated :  
40x53x1 cm
- Seat footprint inflated :  
40x48x25 cm
- Maximum lifting height : 20  
cm
- Portable carrying bag :  
42x54x14 cm
- Weight : 3,7 kg (4,2kg with  
bag)
- Capacity : 120 kg



## RECOMMENDATIONS :

- This equipment is suitable for a person who is independently mobile.
- When you are sitting in the chair, your feet should easily touch the ground.
- Please place the SitnStand as deep as possible in the chair.
- For greater comfort, a pillow can be placed between the back of the chair and the back of the portable lifting cushion.



## PORTABLE LIFTING CUSHION SITNSTAND

The SitnStand is a portable lifting cushion suitable for any type of chair, armchair, or sofa. Lightweight, highly portable, and wrapped in stylish fabric, SitnStand is the best chair lift assist that will enable you to venture outdoors and sit comfortably in nature. Concealed handles make it easy to grip and safely install and move the chair lift assist from place to place, so you can enjoy park benches and other outdoor furniture.

To sit down, sit comfortably in the centre of the cushion and press the arrow ↓. The seat will slowly deflate.

To stand up, release the back from the backrest, place the feet close to the seat, spread them apart and press the arrow ↑ to start the inflation of the seat. When the desired position is reached, you can stand up.



	Electric lift chair 	Portable lifting cushion SitNStand 
Can it be carried outside?	NO	YES
Suitable for any style of chair ?	NO	YES
Operates with a Battery ?	NO	YES
Is it a medical device?	NO	YES
Easy to clean ?	YES	YES
Adjustable lifting height ?	YES	YES

## DIMENSIONS AND WEIGHT :

- Seat footprint deflated : 40x45x6 cm
- Seat footprint inflated : 40x45x20 cm
- Seat width : 45 cm
- Maximum lifting height : 16 cm
- Portable carrying bag (option) : 42x54x14 cm
- Weight : 3,5 kg (4kg with box)
- Capacity : 120 kg

## WHEELCHAIR LIFTING CUSHION SITnSTAND

Wheelchair lifting cushion SitnStand is a sit-to-stand lift chair for wheelchair users. This easy, simple, and safe-to-use chair lift assist is patent pending and suitable for any type of wheelchair and was designed for transferring a person to and from a wheelchair as seamless and independent as possible.



Easy to install, you just need to place the folded SitnStand on the wheelchair seat and slide the back section (the one containing the power unit) between the backrest and the seat. Then place the straps of the back part on the handles of the wheelchair and tighten them.



Without the SitnStand



With the SitnStand



To sit down, lock the wheelchair brakes, sit comfortably in the center of the cushion and press the arrow ↓. The seat will slowly deflate.

To stand up, lock the wheelchair brakes, remove your feet from the footrest and place them next to the chair, stand up straight, spread your feet apart and press the ↑ arrow to start the seat inflation.



## SITN STAND ACCESSORIES



**Additional  
carrying bag**



**Additional  
seat cover**



**Additional form  
memory cushion  
(45x40x4cm)**



**Additional  
battery**



**Additional  
charger**

Watch our demonstration video :



Watch all tutorial videos :

- Getting started
- Sitting down and standing up
- Replacing and charging the battery

With this QR code :



**AXSOL**  
**EXPORT**

45, avenue Georges Politzer - 78190 TRAPPES - France  
Tél. : +33 1 85 83 02 10 - Email : info@axsol-export.eu

[www.axsol-export.eu](http://www.axsol-export.eu)